

## Up and About in Bed.



**Liko™ RepoSheet™ Original**  
**Liko™ Solo™ RepoSheet™**

**Hill-Rom®**

Enhancing Outcomes for Patients and Their Caregivers.™

# RepoSheet™ Saves Backs.



Everybody knows that it is almost impossible to lie still in bed — not even when we sleep do we lie still. Even severely disabled persons sometimes end up wrongly positioned in the bed. Consequently, lifting and repositioning patients is one of the most common operations around a bed. Research\* also shows that many work-related injuries occur here.

When a patient is to be moved higher up in the bed or turned, Liko™ RepoSheet™ is an excellent aid. The Liko™ RepoSheet™ is placed under the patient and can be used both in combination with and instead of an ordinary sheet during the entire care period. (Liko™ Solo™ RepoSheet™ is always used together with an ordinary sheet).

With Liko™ RepoSheet™ and an overhead lift, it becomes possible for just one caregiver to reposition a patient in bed.

## Product Overview


| Product                   | Fabric           | Size    | Prod. No. | Max. Load |
|---------------------------|------------------|---------|-----------|-----------|
| Liko. RepoSheet. Original | Cotton/Polyester | Regular | 3687701   | 200 kg    |
| Liko. RepoSheet. Original | Polyester HT     | Regular | 3687301   | 200 kg    |
| Liko. RepoSheet. Original | Cotton/Polyester | Ultra   | 3687702   | 500 kg    |
| Liko. RepoSheet. Original | Polyester HT     | Ultra   | 3687302   | 500 kg    |

Laundering: 85 °C (185 °F), dry in cabinet dryer or tumble-dry.

| Product                         | Fabric           | Size  | Prod. No.  | Max. Load |
|---------------------------------|------------------|-------|------------|-----------|
| Liko. Solo. RepoSheet. 10 pcs   | Cotton/Polyester | Short | 36871003   | 200 kg    |
| Liko. Solo. RepoSheet. 2x10 pcs | Cotton/Polyester | Short | 36871003-2 | 200 kg    |

Laundering: ~~X~~ Non washable. Can be recycled by incinerating.

**Contact your Liko representative for advice and information about product changes.**

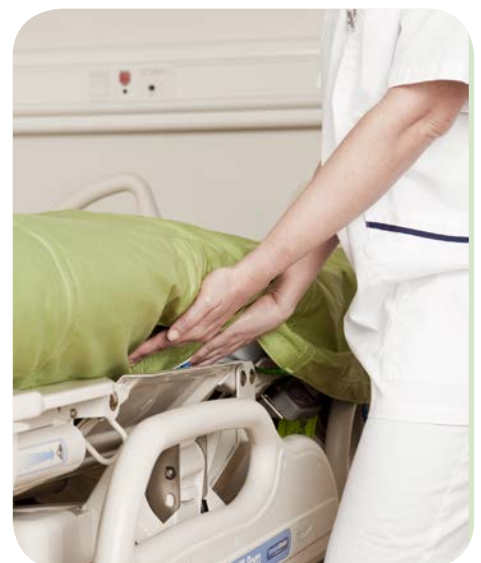

\* *The accident process preceding overexertion back injuries in nursing personel* by Inga-Lill Engkvist, BSc, Mats Hagberg MD, Eva Wigaeus Hjelm, PhD, Ewa Menchel, PhD, Lena Ekenvall MD, PROSA study group. Scand J Work Environ Health 1998;24(5):367-375.

*A dynamic 3D biomechanical evaluation of the load on the low back during different patient-handling tasks* by Skotte JH, Essendrop M, Hansen AF, Schibye B. J Biomech. 2002 Oct;35(10):1357-66.



# Liko™ RepoSheet™ Original

Liko™ RepoSheet™ Original is available in two versions: Regular, for lifting up to 200 kg, and the sturdier Ultra, which manages up to 500 kg. It is true that transfers can be done by one person only, but we recommend two caregivers. It is smoother and timesaving as well. Liko™ RepoSheet™ Original works best in combination with an overhead lift.

## Up In Bed

### Earlier




Manually lifting a patient higher up in bed is heavy and strenuous work.

It is an operation associated with risks and often repeated many times during the day for each patient.



Whoops, the patient has slid down in bed. It is high time to lift him up with the RepoSheet™. The overhead lift is positioned above the patient and the strap loops are attached to the slingbar.

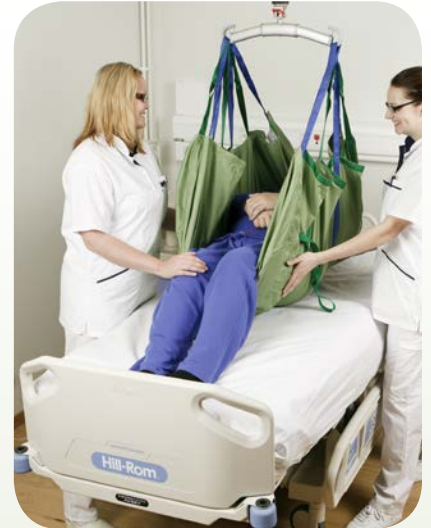

When the patient has been lifted from the bed, he is easily moved higher up with the help of the RepoSheet™ – all with minimal physical effort.

## About In Bed

### Earlier



Many patients have to be turned several times a day to avoid, for instance, the risk of pressure ulcers. Even with a well performed manual technique, caregivers are exposed to heavy loads. There is no getting away from this.



First, lift the patient to one side of the bed with the RepoSheet™.

Then, reconfigure the strap loops.



With the help of the overhead lift, turn the patient to the side in a careful and controlled manner.

# Liko™ Solo™ RepoSheet™

The Liko™ Solo™ RepoSheet™ is a variant of the RepoSheet™, intended for short-term use. The Liko™ Solo™ RepoSheet™ should not be washed, but should be disposed of when soiled or when the patient no longer needs it.



Liko™ has a wide range of overhead lifts. Liko™ is one of several good alternatives in combination with RepoSheet™.



Liko™ RepoSheet™ Original is easily placed in the bed. It is soft to lie on and can be used in combination with and instead of an ordinary bottom sheet.



Support the patient with pillows at suitable places, then detach the Liko™ RepoSheet™ Original and place it in the bed.



Liko™ Solo™ RepoSheet™ is placed on top of the ordinary bed sheet and can be used during the entire care period. Since laundering is eliminated, the Liko™ Solo™ RepoSheet™ solution is relatively economical, while necessary aids can be held available in storage.



Using the Liko™ Solo™ products from Hill-rom, helps to prevent the spread of viruses and contagious bacteria, e.g. MRSA.

## Understand, Develop and Adapt.

Lifting a person requires both good technical aids and knowledge of medical complications. Liko™ is a specialist in lifting and transferring disabled people. For more than 30 years we have successfully developed and manufactured functional aids which are safe and easy to use. A big part of our success is founded precisely on our combination of high technical competence, a medical know-how, and our own experience in health care.

RepoSheet™ is a good example of Hill-Rom® and Liko's ability to understand, develop and adapt products to different needs within health care.

Hill-Rom and design and Hill-Rom word mark are trademarks or registered trademarks of Hill-Rom Services Inc.  
Liko and design and Liko word mark are trademarks or registered trademarks of Liko R&D AB.  
RepoSheet Original, Solo RepoSheet and Likorall are trademarks or registered trademarks of Liko R&D AB.



Design and Quality by Liko in Sweden.

Liko is quality certified in accordance with ISO 9001 and its equivalent for the medical device industry, ISO 13485.  
Liko is also certified in accordance with environmental standard ISO 14001.



Enhancing Outcomes for Patients and Their Caregivers.™

France \_\_\_\_\_ +33 (0)2 97 50 92 12  
United Kingdom \_\_\_\_\_ +44 (0)1530 411000  
Deutschland \_\_\_\_\_ +49 (0)211 16450-0  
Nederland \_\_\_\_\_ +31 (0)347 32 35 32  
Italia \_\_\_\_\_ + 39 (0)2 950541  
Suisse/Schweiz \_\_\_\_\_ +41 (0)21 706 21 30  
(deutschsprachig) \_\_\_\_\_ +41 (0)21 706 21 38

Österreich \_\_\_\_\_ +43 (0)2243 28550  
Ireland \_\_\_\_\_ +353 (0)1 413 6005  
Iberia \_\_\_\_\_ +34 (0)93 6856000  
Nordic Countries \_\_\_\_\_ +46 (0)20 78 10 30  
Export \_\_\_\_\_ Contact your local Hill-Rom distributor or contact your Area Manager via website or call \_\_\_\_\_ +1 812 934 8173

[www.hill-rom.com](http://www.hill-rom.com)  
[www.liko.com](http://www.liko.com)

Hill-Rom® reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom® makes is the express written warranty extended on the sale or rental of its products.

© 2012 Hill-Rom® SARL, ALL RIGHTS RESERVED.